




STREPTO GROUP TEST STAPHYLO RAPID TEST

OUR IN VITRO
DIAGNOSTICS
IDENTITY

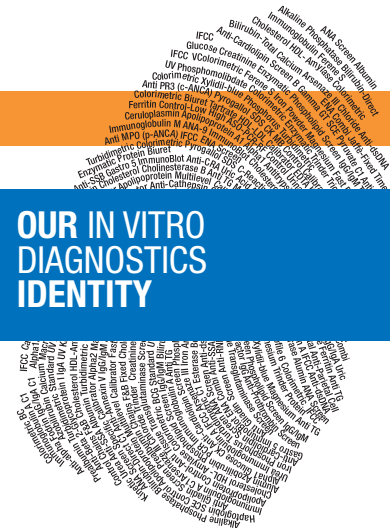

LATEX AGLUTINATION TEST FOR THE IDENTIFICATION

Slavo, as a reference partner for the clinical laboratory with a strong experience in In Vitro Diagnostics, offers a complete line of reagents devoted to the study in Bacteriology.



ADVANTAGES

STREPTO GROUP TEST

- **rapid**
the positive reaction is generally evident after 15-30 seconds
- **accurate**
the sensitized reagents are specific for the single groups and do not give any cross reactions
- **easy interpretation**
agglutination of the white latex on the black reaction slide facilitates reading of the results
- **complete**
the kit contains all the materials necessary for the execution of the test, including positive and negative controls

STAPHYLO RAPID TEST

- **rapid**
the positive reaction becomes evident with most strains after only 5 seconds
- **accurate**
highly sensitive even with Methicillin-resistant *S. aureus* (MRSA)
- **easy interpretation**
agglutination of the white latex on the black reaction slide facilitates reading of the results

**STREPTO GROUP TEST
STAPHYLO RAPID TEST**



**OUR IN VITRO
DIAGNOSTICS
IDENTITY**

PRINCIPLES

STREPTO GROUP TEST

Classification of streptococci according to Lancefield, on the basis of specific polysaccharide anti-
gens, after chemical extraction from the colony.

The test utilizes the agglutination between the specific streptococcal antigen and the specific serum
adsorbed on the latex. If the sample shows a negative reaction with the latexes of Groups A, B, C, D,
F and G, enzymatic extraction is performed and tested with the Group D latex.

KIT PRESENTATION

CSI087315	Strepto Group Test	300 tests
-----------	--------------------	-----------

SINGLE COMPONENTS

CSI087331	Strepto Group Latex Test A	50 tests
CSI087332	Strepto Group Latex Test B	50 tests
CSI087333	Strepto Group Latex Test C	50 tests
CSI087334	Strepto Group Latex Test D	50 tests
CSI087335	Strepto Group Latex Test F	50 tests
CSI087336	Strepto Group Latex Test G	50 tests
CSI087337	Strepto Group Test Extracting Reagent E	50 tests
CSI087339	Strepto Group Test Chemical Extracting Reagents	50 tests
CSI087338	Strepto Group Test Positive Control	1 ml

STAPHYLO RAPID TEST

The kit detects the bound coagulase (Clumping Factor) and Protein A present on the surface of
S. aureus.

KIT PRESENTATION

CSI087245	Staphylo Rapid Test	50 tests
-----------	---------------------	----------



SCLAVO DIAGNOSTICS INTERNATIONAL SRL
VIA PO 26-28 LOCALITÀ PIAN DEI MORI
53018 SOVICILLE SIENA (ITALY)
PH +39 02 93 991 569 FAX +39 02 93 991 393

sclavoexport@dggroup.it
www.sclavodiagnostics.com